Aesthetic
MEDICAL PRACTITIONER

SUCCESS HAS MANY FACES

Over dionisp are disvormer dictions per disvormer dictions pe

PRODUCTS



With one handpiece and four interchangeable tips, this patented radiofrequency (RF) device is indicated to treat the face, eyes and body. The new Total Tip 4.0 covers 33 per cent more surface area than the Total Tip 3.0, making treatment times 25 per cent faster.

Thermage FLX boasts a light, ergonomically designed handpiece with multi-directional vibration and a large touchscreen with a user-friendly interface, making the system easier to navigate than its predecessors. Enhanced face and body vibration helps maintain patient comfort during treatments.

HOW IT WORKS

Thermage FLX delivers monopolar RF energy to the surface of the skin via cryogen-cooled tips attached to a handpiece. This technology heats the dermal tissue while protecting the more superficial layers of the epidermis and dermis.

VVE HAVE HAD A SUCCESS RATE OF OVER 95 PER CENT WITH OUR THERMAGE TREATMENTS 9

- Dr Buddy Beaini

Treatments last 30-90 minutes depending on the treatment area. Some patients have temporary redness or minor swelling after treatment, but for most the Thermage FLX treatment requires zero downtime.

Unlike many cosmetic procedures, Thermage FLX delivers results after a single treatment. These results gradually appear two to six months post-treatment and, depending on the skin condition and ageing journey of the patient, can last years. Thermage FLX delivers impressive, easy to predict results and has been found to improve facial and skin laxity in 96 per cent of patients.

ABOUT ACCUREP™

Previous versions of Thermage like Thermage CPT required tuning at the beginning of each treatment session. This average impedance measurement would then be used for all subsequent therapeutic radiofrequency energy pulses (REPs). While reasonably effective, this method can lead to some variability in the amount of energy delivered to the target tissue when local impedance at the treatment site is not the same as the average impedance value to which the system was tuned prior to treatment.

The Thermage FLX system uses AccuREP™ technology to integrate tuning pulses into each individual REP. These tuning pulses automatically measure local impedance values and adjust energy output parameters to deliver consistent energy from one treatment site to the next.

With a reputation for delivering noticeable results from a single comfortable treatment, Thermage by Solta Medical is a leader in non-invasive skin tightening technologies. Now, with the addition of AccuREP™ and the new Total Tip 4.0, Thermage FLX is inspiring aesthetic medical practitioners and their patients in more ways than ever before.

PATIENTS HEARD
ABOUT THERMAGE
FLX, THEY WERE
EAGER TO TRY IT AS
THERE WAS AN
INHERENT TRUST IN
THE BRAND. 9

- Sylvia Down



AS SEEN IN AESTHETIC MEDICAL PRACTITIONER



PRODUCTS



SYLVIA DOWN Practice Manager, Skin Renu

What attracted you to Thermage FLX?

When Skin Renu opened 11 years ago, Thermage immediately became one of our top performing treatments as it consistently delivered great outcomes.

Naturally, we were excited to hear about the launch of the next generation Thermage system, a sleek new platform developed with game-changing improvements. The bio-feedback provided by the newly added AccuREPTM technology really made Thermage FLX stand out.

Who is your typical Thermage FLX patient?

A typical Thermage FLX patient is someone who understands the need to care for their skin at a deeper. As a one-off treatment performed biannually or annually with no downtime, Thermage FLX appeals to a variety of patients.

Our younger patients see
Thermage FLX as an effective way
of delaying the first visible signs
of ageing. Many young mothers
choose Thermage FLX for treating
post-baby lax tummy skin. Our
more mature patients turn to
Thermage FLX for the tightening
and lift it provides.

What kind of results has Skin Renu achieved with Thermage FLX?

Thermage FLX has been delivering a more noticeable immediate effect than the previous

generation. We will stop the treatment after completing one side of the face or body and allow the patient to compare the two sides. We recently treated a 39-year-old female who was concerned with the lax skin above her knees. When we showed her the results on the first leg she was amazed.

I believe the best measure of Thermage FLX, and indeed all Thermage, is the loyalty of our clients. When existing patients heard about the development of Thermage FLX, they were eager to try the new technology as there was an inherent trust in the brand.

Do you use Thermage FLX in combination with any other treatments?

Thermage FLX works beautifully with a range of other treatments, from Clear + Brilliant to Fraxel, INFINI to PICO Genesis, to all injectables.

How does Thermage FLX benefit your practice?

Since satisfaction and trust are the cornerstones of any business, by delivering on its promises Thermage FLX allows us to build strong relationships with our patients. The people behind Solta are the frontline and face of the company. There is always someone available to help with enquires. Angie Meredith, our BDM, is efficient, professional and an absolute gem. **AMP**

For more information call 1800 251 150 or visit soltamedical.com.au